
# Retreatment

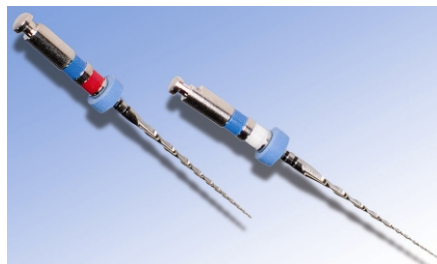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### Limas de retratamiento Mtwo 15/05 y 25/05

Velocidad recomendada 280 r.p.m.

Lima 15/05: torsión máx. 30 gcM (0,3 NcM).

Lima: 25/05: torsión máx. 120 gcM (1,2 NcM).

Mtwo direct: R15/05 Ajuste A. R25/05 Ajuste C

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Max. Torque 25/05 120 gcM (1,2 NcM)

Mtwo direct: R15/05 Setting A. R25/05 Setting C.

"Open the canal orifice with a gates glidden drill or an ultrasound instrument and remove guttapercha from the coronal third. If necessary, soften guttapercha with a solvent (e.g. eucalyptus oil).

"Place the tip of the retreatment instrument on the guttapercha and allow it to rotate.

Mtwo retreatment files have a cutting tip so that the instrument should progress easily in the guttapercha without pressure.

"Use Mtwo Retreatment instruments without downward pressure.

"Gradually remove the root canal filling with circumferential filing movements.

"Use a hand instrument (e.g. C-Pilot file) size ISO 08 to negotiate the root canal to full working length.

"Once full working length to the apex has been negotiated with a small, flexible hand instrument, prepare the root canal to size ISO 15 to full working length.

"Continue the preparation with conventional Mtwo instruments until the required size has been reached.

"Preparation example:

Mtwo R15/05 or Mtwo R25/05, hand instruments ISO 08, ISO 10, ISO 15, Mtwo 10/04, Mtwo 15/05, Mtwo 20/06, Mtwo 25/06 etc.

For optimum safety, use a NiTi instrument only once.

Always check instruments for signs of wear. As a general guide, use an Mtwo retreatment instrument in 3-4 canals if guttapercha is more difficult to remove and in 6-8 canals if guttapercha is easy to remove.